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iCARE User's Manual

## iCARE User's Manual


The iCARE box contains : one 8 in 1 measurement device, one micro-usb charging cable, and one protective box.

### Cloudmed iCARE Disclaimer

Applicable age is 18 - 75 years old. This product is for self-health management only. All the service/information congregated from iCARE and iCARE APP is not intended for use in the diagnosis of any medical condition or cure, treatment, or prevention of disease. The performance and accuracy of the information and measured data by iCARE and iCARE app may be affected by skin-condition, environmental condition, personal profile, iCARE settings, and user's gesture.

Please follow the given instructions to setup your iCARE device with iCARE APP

## 1. Product Introduction

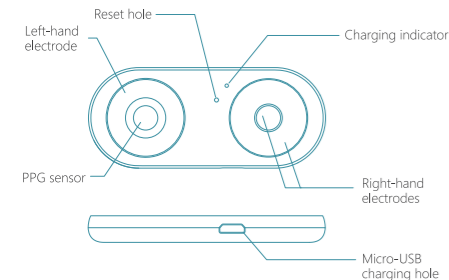
## 2. Install iCARE APP on Phone

Search "CloudMed iCARE" APP on Google Play/Apple Store or scan the QR Code and download the app from official website.


## 3. To Connect via Bluetooth

Start iCARE APP and follow the instructions to create an account. Then, click the Bluetooth icon to search iCARE device and start measuring.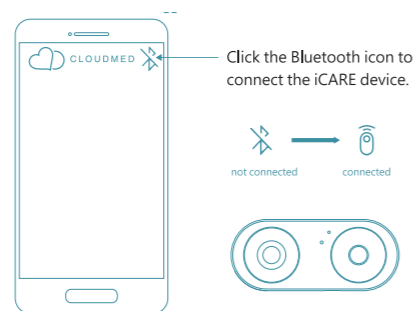


## 4. iCARE Charging

When the battery capacity is low, please charge the iCARE device. It's not recommended to use the iCARE device during charging due to the power-line interference.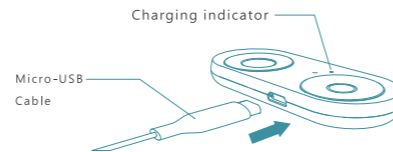

## 5. Measurement Reminder

Please put your fingers on the upper side of the device and relax yourself during the measurement.

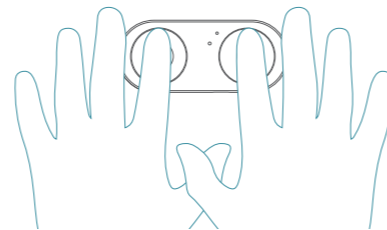## 6. Six Features

8 in 1 Quick Measurement (Within 60 seconds.)

Instant Report with Suggestions

Health Journal in Cloud (Free for 3 months health data storage.\*)

Waveform Recording

Medication Reminder

One Click for SOS+Positioning

\* Cloudmed Co.,Ltd. reserves the right to make changes.

## 7. Specifications

Product	Information
Product Size	69.2x29.5x7.5 mm
Battery Capacity	75 mAh
System Requirement	BLE 4.2 or above
Input Voltage	5.0 V (DC)
Charging Function	Yes(Micro-USB)
Charging Indicator	Yes/Red
Phone Version	Android 6.0 / iPhone 7 or above

## 8. Service Policy

1. The Cloudmed iCARE comes with an one year warranty from date of purchase.

2. Only hardware and manufacturing defects are covered under warranty.

3. Contact Cloudmed helpline through voice, email or chat for any issues with your device.

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